

DURABLE & STRONG

Assist patients or loved ones with confidence in our product.



9503 Series
Gait Belt W/Metal Buckle

HOW TO APPLY A GAIT BELT

1

Place the belt around patient's waist with the tag facing outward and the buckle centered.

2

Thread the metal-tipped end through the buckle hinge over the teeth and tighten.


3

Thread the metal-tipped end through the front buckle opening.

4

Pull tight at each end of the webbing to ensure the belt is secure.

SIZE GUIDE

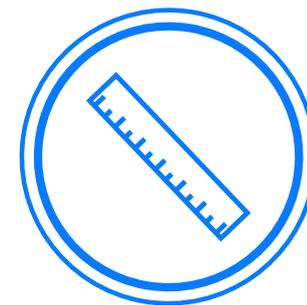

SIZE **FITS UP TO**

S

54"

M

60"

L

72"

HOW TO CHOOSE YOUR SIZE

Using a soft tape, measure the circumference around the widest part of your body where you plan to wear the gait belt.





MACHINE WASHABLE

Wash lukewarm and dry on low to maintain pristine condition.



CHOICE OF BELT COLORS

NATURAL

PASTEL

CORAL

FIESTA

LAVENDER

NAVY

**STARS &
STRIPES**

